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Central supply units and supply network

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Heyer Aerotech is a manufacturer of medical equipment and on our premises in Nievern/Germany we develop, make and distribute innovative and high-quality equipment for medical gas supply as well as medical accessories which are regarded today as indispensable standards in modern health-oriented facilities, hospitals, clinics, sanatoria and medical practices.

As a manufacturer with our own production site and with a high vertical range of manufacture, we make our products exclusively on our premises in Nievern on the basis of certified quality and risk management systems which comply with national and international standards. High-quality technologies as well as accuracy in long-sighted and thorough planning are taken for granted.

Our philosophy of thinking globally and acting locally is determined by our joint efforts in research and development and our cooperation with innovative companies worldwide. In addition, we regard continuity, close customer relationships, high quality and promoting the know-how of our members of staff as the pillars of this philosophy. And most importantly, we always keep a close eye on our main objective: ultimate customer satisfaction based on individual and all-embracing system solutions. After installation of the project, our trained and experienced service team all over Germany takes care of our customers – for a competent, fast and uncomplicated service.

You as a planner, decision-maker or responsible operator of a medical facility may profit from our competence, precision and reliability.

Please feel free to contact us at any time. We will be pleased to support you with our expert advice.

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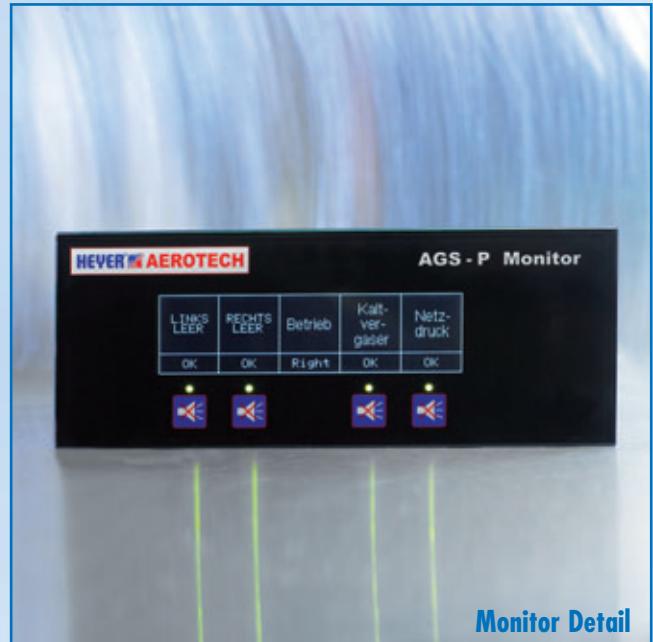
- Changeover unit for regulation, supervision and control of medical and technical gas supply units
- With pneumatically-controlled changeover
- All pneumatic components mounted on a board and protected by means of a plastic cover with door
- Pressures readable from outside
- AGS P monitor with an innovative glass display for indication of the current status of the unit and for electronic supervision of the medical gas supply
- Operation by means of touch keys
- Possibility of passing information on to an external communication / indication system by means of potential-free contacts
- Communication of several Medium Monitors possible via bus system

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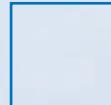
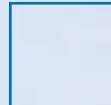
Fully-automated pressure reducer station AGS 80 · Performance up to 80 Nm³/h

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- Higher ranges of performance upon request (up to 160 Nm³/h)
- Changeover unit for regulation, supervision and control of medical and technical gas supply units
- With pneumatically-controlled changeover
- All pneumatic components mounted on a board and protected by means of a plastic cover with door
- Pressures readable from outside
- AGS P monitor with an innovative glass display for indication of the current status of the unit and for electronic supervision of the medical gas supply
- Operation by means of touch keys
- Possibility of passing information on to an external communication / indication system by means of potential-free contacts
- Communication of several Medium Monitors possible via bus system
- Digital pressure indication optional
- Flow measurement optional

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Design with 2 mains pressure reducers and connection for 3 supply sources

	O2 O2	Air Air	N2O N2O	CO2 CO2	
Artikel-Nr.	Article no.	538-3101	538-3106	538-3111	538-3121
H x B x T	H x W x D	750 x 765 x 305 mm			
Gewicht	Weight	60 kg			

Vollautomatische Druckmindererstation AGS 80 D - Leistung bis 80 Nm³/h
Fully-automated pressure reducer station AGS 80 D - Performance up to 80 Nm³/h

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- Changeover unit for regulation, supervision and control of the medical and technical gas supply
- The control is taken over by a 32 Bit microcontroller
- The switch over between the supply sources (cylinders) is taken over by magnetic valves
- In case of power failure the functionality is given because of the fact that the magnetic valves are normally-open (NO)
- If the main power supply fails, a battery will take over the power supply and ensure that the electronics of the system remain in operation.
- The AGS D Monitor is made of an innovative glass display which enables you to see the current status of the unit and to have the electronic supervision of the medical gas supply.
- Operation by means of touch keys
- Possibility of passing information to an external communication / indication system (via potential-free contacts)
- Communication of several Medium Monitors possible (via bus system)
- Flow measurement (optional)
- Higher ranges of performance – upon request (up to 160 Nm³/h)

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Reduzierstationen Reducer stations

Reduzierstation für Flaschenbatterie EGS 35-50
Reducer station for cylinder bank EGS 35-50

Reduzierstation für Flaschenbatterie EGS 80
Reducer station for cylinder bank EGS 80

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
- 2-stage pressure reduction to the required mains pressure
- Contact manometer for supervision of the cylinder pressure
- Applicable as 3rd supply source for cylinder supply
- Exit soldered connection Ø 15 mm or 22 mm

		O2 O2	Air Air	N2O N2O	CO2 CO2	O2 O2
Artikel-Nr.	Article no.	537-3017	531-3021	537-3018	537-3022	410-0060
Normvolumenstrom	Standard volume flow	35 bis 50 Nm³/h				80 Nm³/h
H x B x T	H x W x D	380 x 420 x 220 mm				350 x 430 x 260 mm
Gewicht	Weight	8 kg				13,5 kg

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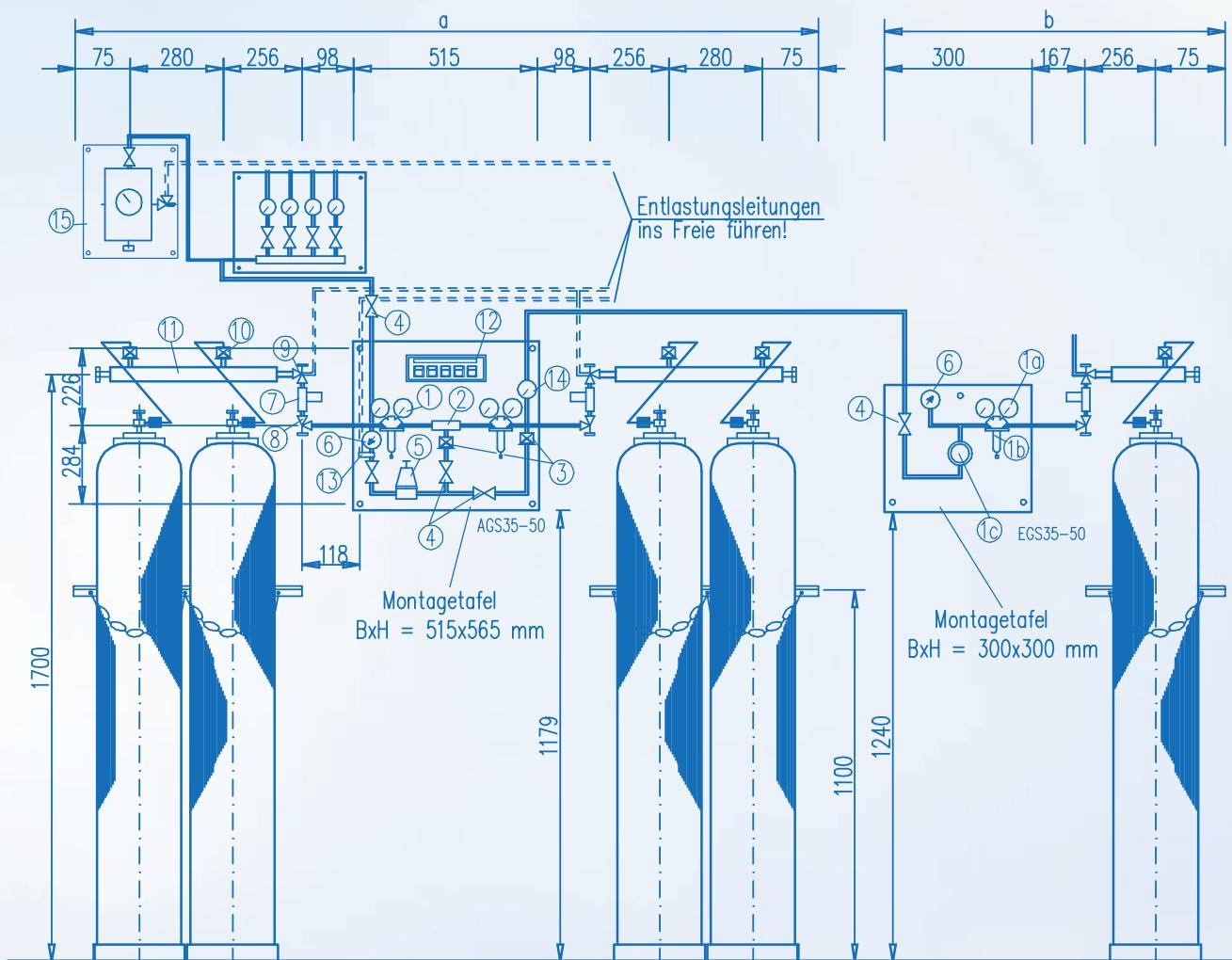Assembly drawing AGS 35 – 50 with 3rd supply source


	a
2x1 Flasche	ca. 1435 mm
2x2 Flaschen	ca. 1985 mm
2x3 Flaschen	ca. 2535 mm
2x4 Flaschen	ca. 3085 mm
2x5 Flaschen	ca. 3635 mm
2x6 Flaschen	ca. 4185 mm

	b
1x1 Flasche	ca. 800 mm
1x2 Flaschen	ca. 1100 mm
1x3 Flaschen	ca. 1400 mm
1x4 Flaschen	ca. 1700 mm
1x5 Flaschen	ca. 2000 mm
1x6 Flaschen	ca. 2300 mm

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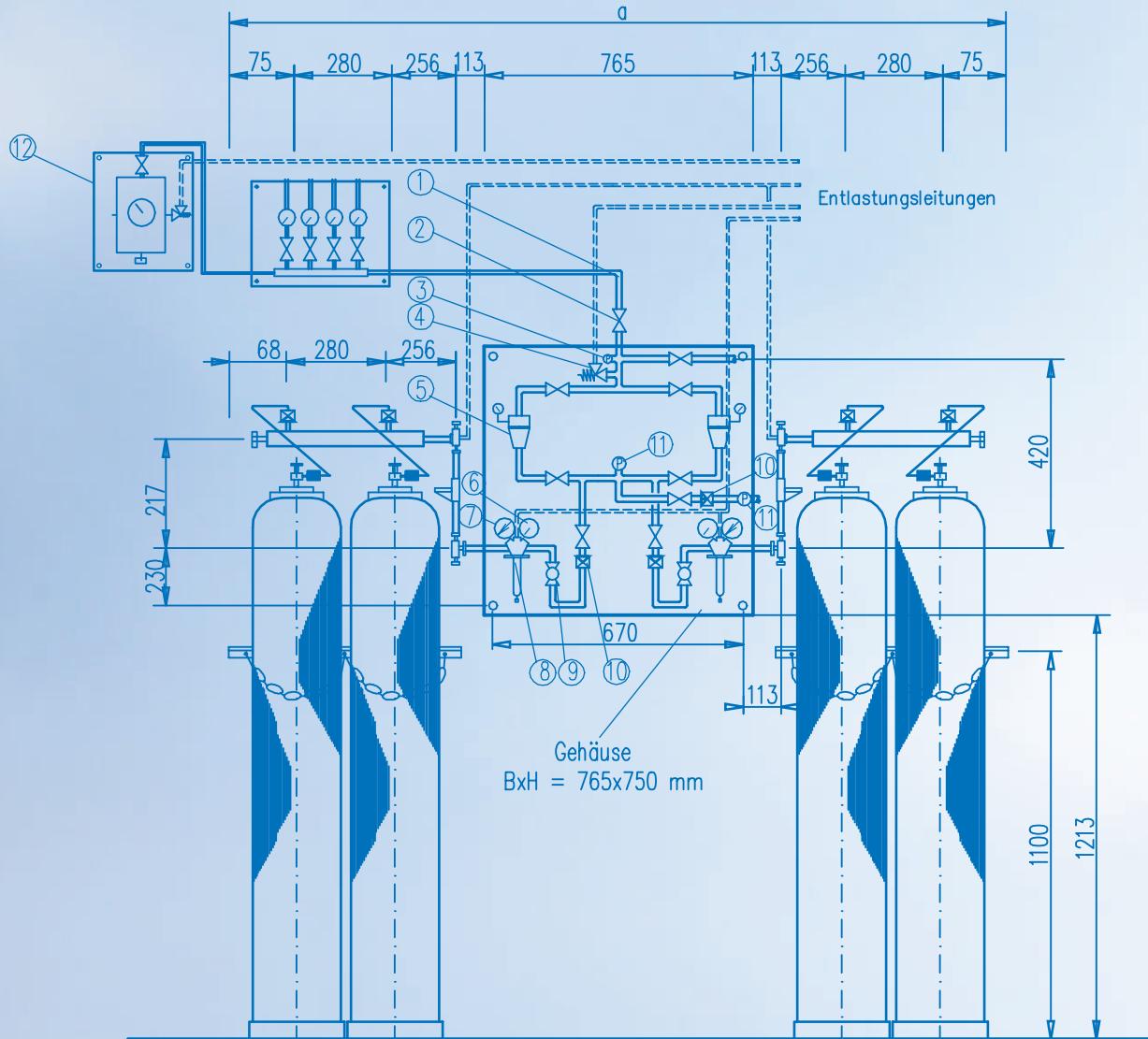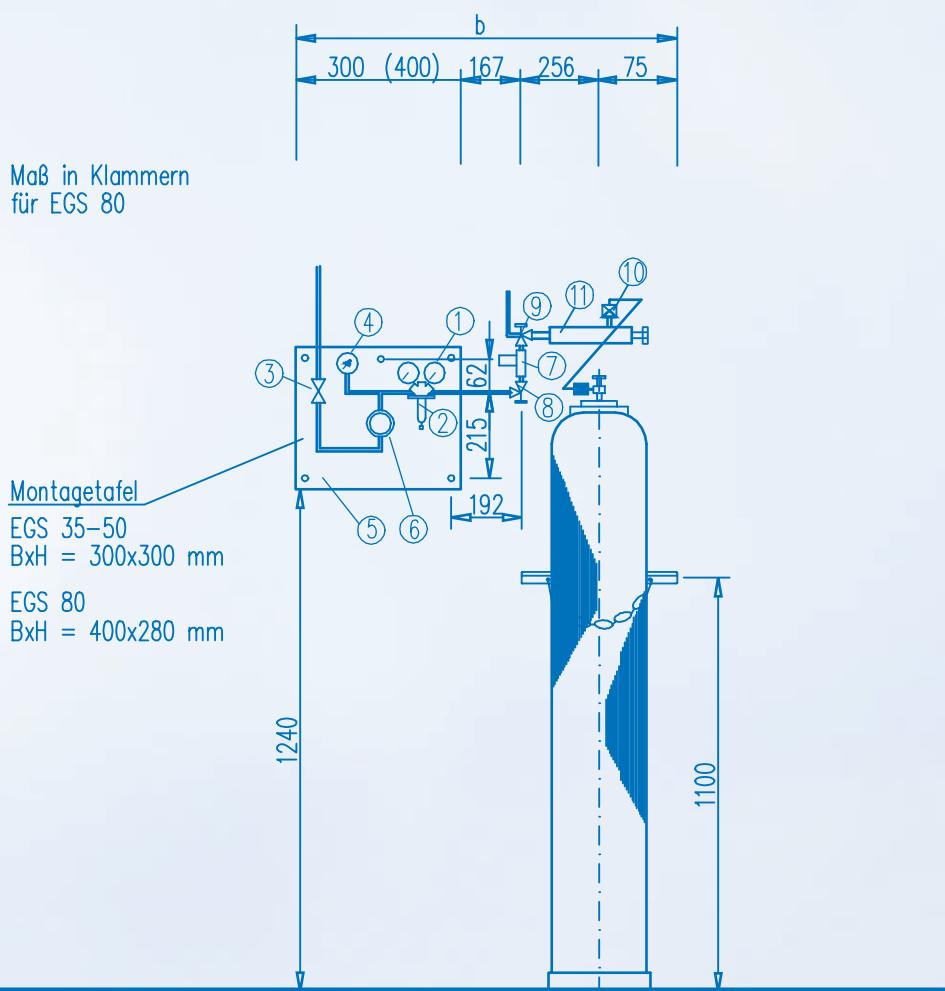
- For pressure reduction to the required system pressure for a switchover unit
- Equipped with a pressure reducer and a sintered metal filter

	Cu-Rohr Ø 22 mm Copper pipe	Cu-Rohr Ø 28 mm Copper pipe
Artikel-Nr. Article no.	536-3002	536-3003
Nm ³ /h bei p2 = 5 bar Nm ³ /h at p2 = 5 bar	450	
H x B x T H x W x D	400 x 500 x 160 mm	
Gewicht Weight	9 kg	10 kg

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- For pressure reduction to the required system pressure for a switchover unit
- Equipped with a pressure reducer (optional) and a sintered metal filter, pressure switch, one-way valve
- With bypass capacity, for maintenance of the components without interruption of the supply

	Mit Druckminderer With pressure reducer	Ohne Druckminderer Without pressure reducer
Artikel-Nr. Article no.	536-3001	536-3000
Cu-Rohr Ø Copper pipe Ø	22 mm	
H x B x T H x W x D	400 x 800 x 160 mm	
Gewicht Weight	14 kg	12 kg

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High-pressure sintered metal filter HP-G

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- High-pressure sintered metal filter up to 250 bar
- Nominal width DN 10
- BAM-tested
- Sintered filter with pore size of appr. 30 µm

		Hochdruck Sintermetallfilter High-pressure sintered metal filter
Artikel-Nr.	Article no.	021-3950
H x B x T	H x W x D	130 x 110 x 50 mm
Gewicht	Weight	1,9 kg

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- Monitoring, backing and maintaining of mains pressure
- Equipped with a contact manometer for monitoring the mains pressure, safety valve, with a gas type-specific feed point (NIST connection)
- Maintenance of components possible without interruption of the supply

	O2 O2	Air 5 bar Air 5 bar	Air 800 Air 800	N2O N2O	CO2 CO2	
Artikel-Nr.	Article no.	538-3485	538-3495	538-3497	538-3490	538-3496
H x B x T	H x W x D	280 x 250 x 90 mm				
Gewicht	Weight	3 kg				

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■ High-pressure release valve – corner design up to 250 bar

Hochdruck Eck-Entlastungsventil High-pressure release valve corner design	
Artikel-Nr.	Article no.
H x B x T	H x W x D
Gewicht	Weight

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■ High-pressure release valve straight up to 250 bar

Hochdruck Entlastungsventil High-pressure release valve	
Artikel-Nr.	Article no.
H x B x T	H x W x D
Gewicht	Weight

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■ High-pressure shut-off valve for operating pressure of 200 bar

		Hochdruck Absperrventil <i>High-pressure shut-off valve</i>
Artikel-Nr.	Article no.	016-3900
H x B x T	H x W x D	130 x 65 x 65 mm
Gewicht	Weight	700 g

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
■ BAM geprüft

■ High-pressure one-way valve up to 250 bar

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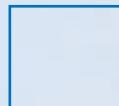■ With outside thread for shut-off valve

	1-fach 1-fold	2-fach 2-fold	3-fach 3-fold	4-fach 4-fold	5-fach 5-fold	6-fach 6-fold
Artikel-Nr. <i>Article no.</i>	548-1000	548-2000	548-2100	548-2200	548-2300	548-2400
L x H x T <i>L x H x D</i>	330 x 100 x 100 mm	560 x 100 x 100 mm	840 x 100 x 100 mm	1120 x 100 x 100 mm	1400 x 100 x 100 mm	1680 x 100 x 100 mm
Gewicht <i>Weight</i>	1,5 kg	2,25 kg	3 kg	3,75 kg	4,5 kg	5,25 kg

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■ With inside thread for one-way-valve

	1-fach 1-fold	2-fach 2-fold	3-fach 3-fold	4-fach 4-fold	5-fach 5-fold	6-fach 6-fold
Artikel-Nr. <i>Article no.</i>	548-1050	548-2050	548-2150	548-2250	548-2350	548-2450
L x H x T <i>L x H x D</i>	330 x 100 x 100 mm	560 x 100 x 100 mm	840 x 100 x 100 mm	1120 x 100 x 100 mm	1400 x 100 x 100 mm	1680 x 100 x 100 mm
Gewicht <i>Weight</i>	1,5 kg	2,25 kg	3 kg	3,75 kg	4,5 kg	5,25 kg

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- For compressed gas cylinders 50 l volume
- Powder-coated white with chain

	1-fach 1-fold	2-fach 2-fold	3-fach 3-fold	4-fach 4-fold	5-fach 5-fold	6-fach 6-fold
Artikel-Nr. Article no.	547-1000	547-1100	547-1200	547-1300	547-1400	547-1500
L x H x T L x H x D	260 x 40 x 40 mm	520 x 40 x 40 mm	755 x 40 x 40 mm	1030 x 40 x 40 mm	1290 x 40 x 40 mm	1550 x 40 x 40 mm
Gewicht Weight	0,4 kg	0,8 kg	1,2 kg	1,6 kg	2,0 kg	2,4 kg

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- Elastic connecting bend with patented hand screw connection
- Material: copper pipe · length 1,000 mm
- Other versions on request

	O2 O2	Air Air	N2O N2O	CO2 CO2	N2 N2
Artikel-Nr. Article no.	546-1750	546-1950	546-1800	546-1900	546-1850

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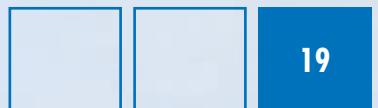
■ High-pressure contact manometer 0 to 400 bar / mark at 200 bar
 ■ Adjustable warning contact
 ■ Various pressure ranges upon request

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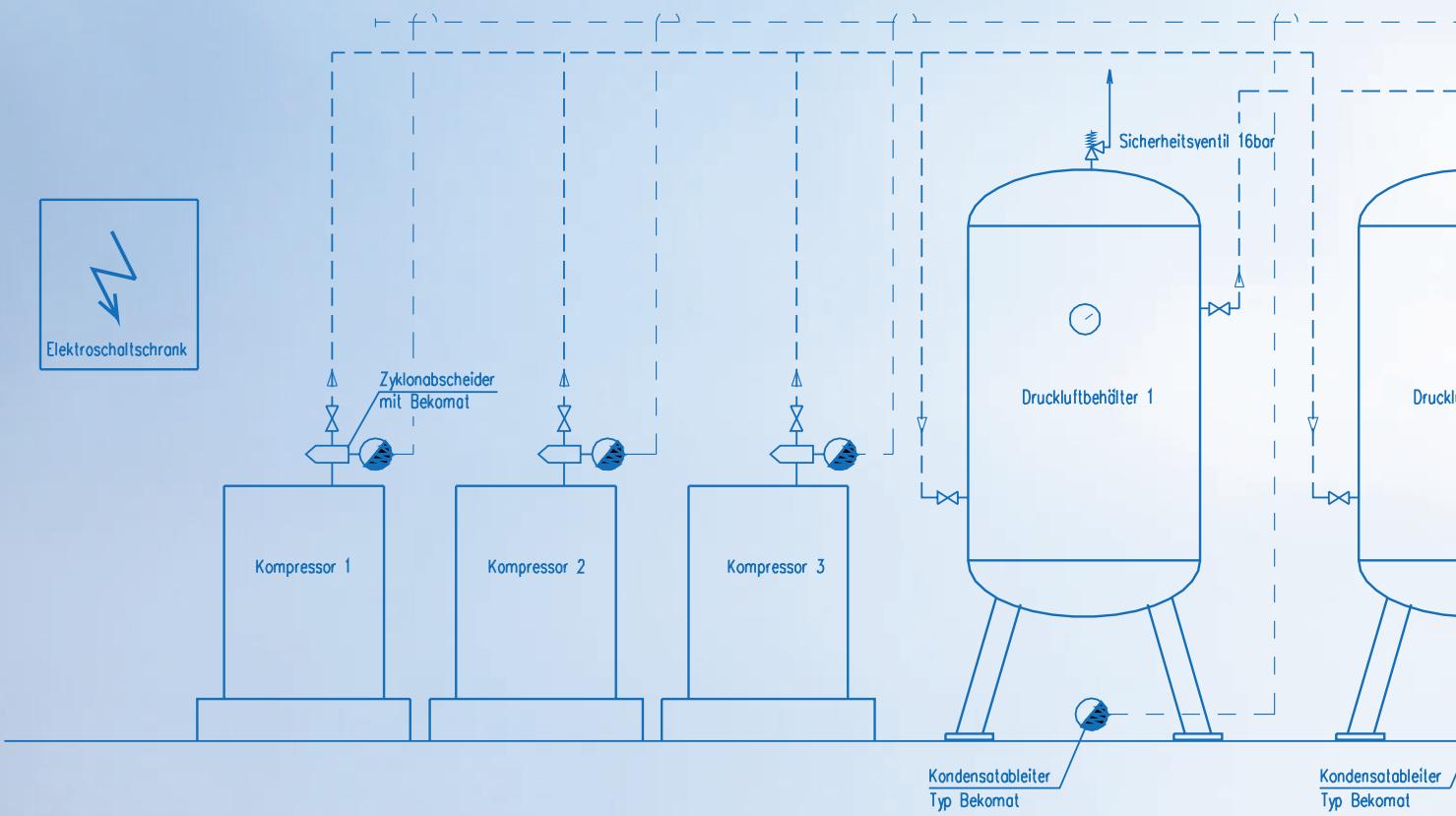
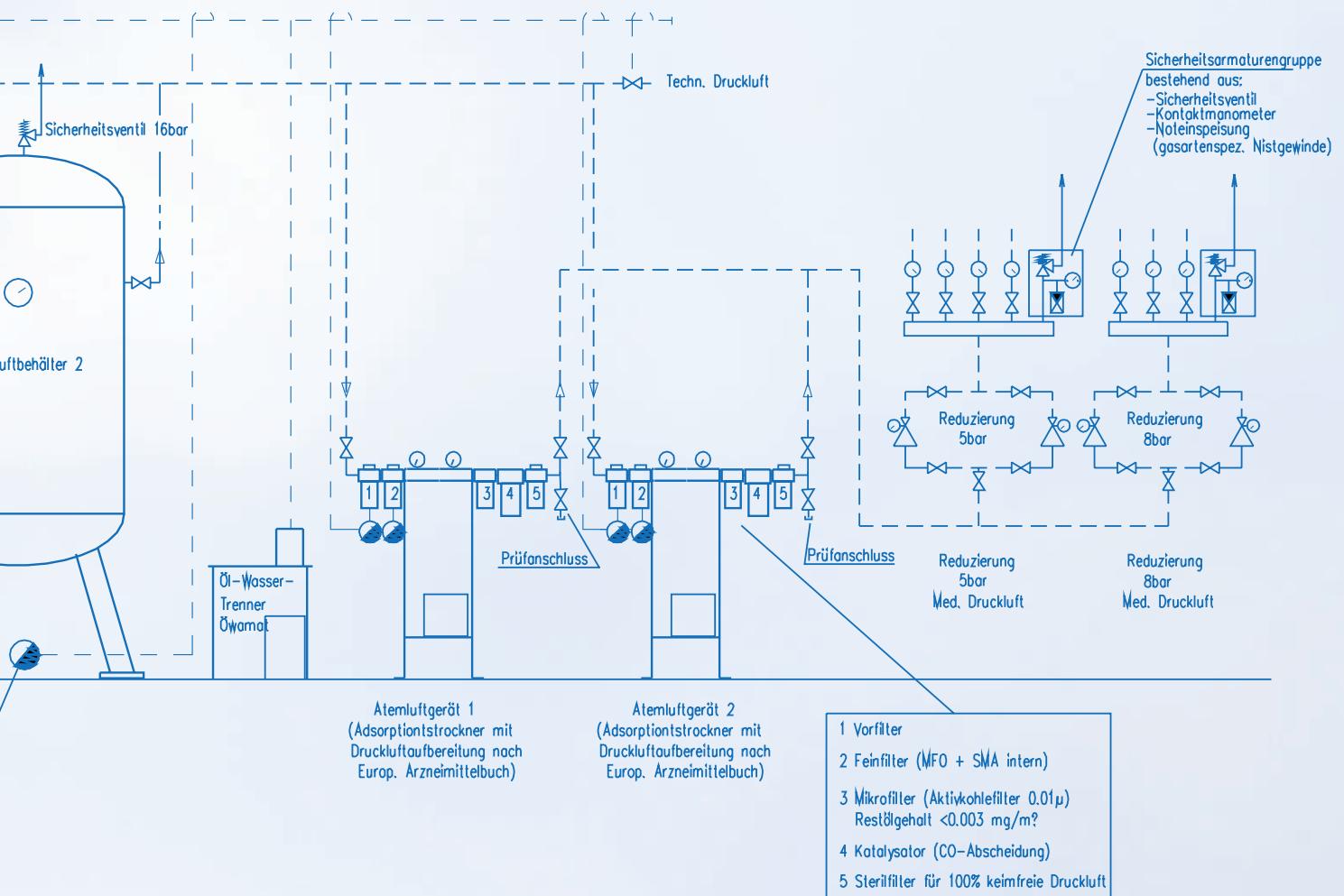
The central supply stations for medical compressed air are designed in accordance with the customers' requirements and the local conditions. The individual components, such as compressors, air pressure tanks, compressed air preparation systems or oil-water separators are adjusted to the individual consumption. In other words, you will get the perfect solution with a high level of user safety, profitability and reliability.

When planning and implementing the project, the rules and regulations valid at the time and place will be taken into account. Therefore the plants provided to the users comply with the latest international standards.

Our range of compression aggregates, compressed air preparation systems etc. includes all renowned manufacturers.

In our factory, the components – specifically adjusted to each individual case of application – are pre-installed on assembly boards.

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Reduzierstation für Druckgase Reducer station for compressed gases

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- For reduction to the required mains pressure
- Redundant setup
- Optional with contact manometer to monitor the mains pressure
- For the design with contact manometers, please indicate additionally Art. no. 520-1955

	Cu-Rohr Ø 12 mm Copper pipe Ø 12 mm	Cu-Rohr Ø 15 mm Copper pipe Ø 15 mm	Cu-Rohr Ø 22 mm Copper pipe Ø 22 mm	Cu-Rohr Ø 28 mm Copper pipe Ø 28 mm	Cu-Rohr Ø 35 mm Copper pipe Ø 35 mm
Artikel-Nr. Article no.	520-1950	520-1951	520-1952	520-1953	520-1954
Nm ³ /h bei p2 = 5 bar Nm ³ /h at p2 = 5 bar	110	150	450	450	500
H x B x T H x W x D	500 x 400 x 100 mm	500 x 400 x 100 mm	500 x 400 x 160 mm	500 x 400 x 160 mm	500 x 400 x 200 mm
Gewicht Weight	15 kg	17,5 kg	20 kg	22,5 kg	25 kg

Adsorptionstrockner Adsorption dryer

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- For preparation of compressed air acc. to Pharmacopee Europeene
- With load-dependent control incl. dew point measuring (optional)
- 5-stage filtering of compressed air
- 2 Adsorber cells for air drying

Typ Type	Leistung* m ³ /h Capacity* m ³ /h	Abmessungen mm Dimensions mm			Gewicht kg Weight kg	Anschluss Connection
		B	T	H		
ATM 15	180	1100	664	1106	191	1"
ATM 18	216	1100	664	1266	212	1"
ATM 22	252	1154	664	1382	222	1"
ATM 34	408	1154	729	1652	301	1 1/2"
ATM 45	576	1527	729	2011	337	1 1/2"
ATM 55	720	1980	875	1800	417	1 1/2"

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

- For preparation of compressed air acc. to Pharmacopee Europeenne
- With load-dependent control incl. dew point measuring (optional)
- 5-stage filtering of compressed air
- 2 Adsorber cells for air drying

Typ Type	Leistung* m ³ /h Capacity* m ³ /h	Abmessungen mm Dimensions mm			Gewicht kg Weight kg	Anschluss Connection
		B	T	H		
ATM-AL 1	6	720	205	550	18	1/4"
ATM-AL 2	12	720	205	685	23	1/4"
ATM-AL 3	24	750	205	895	34	3/8"
ATM-AL 4	42	750	215	1155	42	3/8"
ATM-AL 6	60	885	215	1218	51	1/2"
ATM-AL 8	84	885	215	1360	70	1/2"
ATM-AL 9	108	1005	260	1360	82	1/2"

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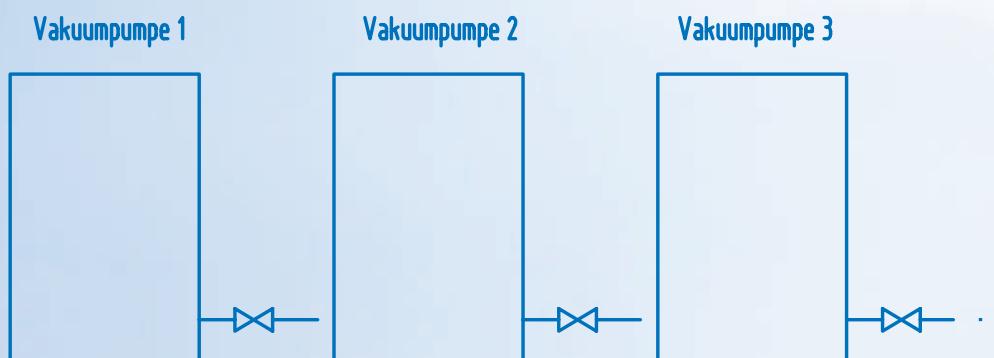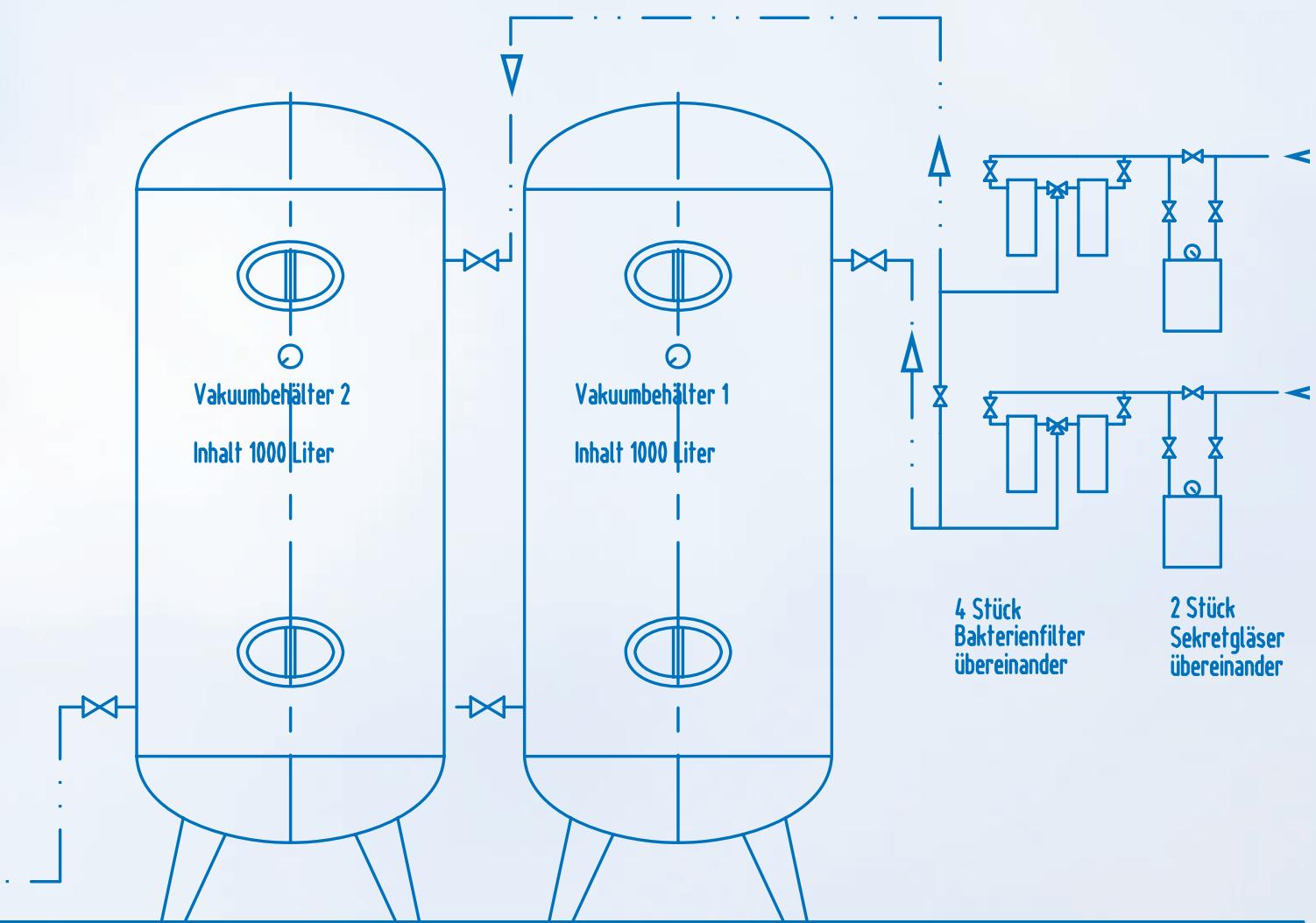The central supply stations for medical vacuum are designed in accordance with the customers' requirements and the local conditions. The vacuum pumps and vacuum tanks are adjusted to the individual consumption. In other words, you will get the perfect solution with a high level of user safety, profitability and reliability.

When planning and implementing the project, the rules and regulations valid at the time and place will be taken into account. Therefore the plants provided to the users comply with the latest international standards.

Our range of vacuum pumps includes all renowned manufacturers. In our factory, the components – specifically adjusted to each individual case of application – are pre-installed on assembly boards.

Vakuumschalter Tafel
Vacuum switch panel

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■ For vacuum pump control

Artikel-Nr. Article no.	H x B x T H x W x D	Gewicht Weight
520-1963	700 x 260 x 120 mm	8,5 kg

Auffangglas & Filter Collecting jar & filter

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Bacteria double-filter for vacuum station · performance up to 200 Nm³/h

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■ Cu-Rohr Ø 35 mm

■ Performance up to 200 Nm³/h – Higher ranges of performance upon request

■ With switching valve for interruption-free change of filters

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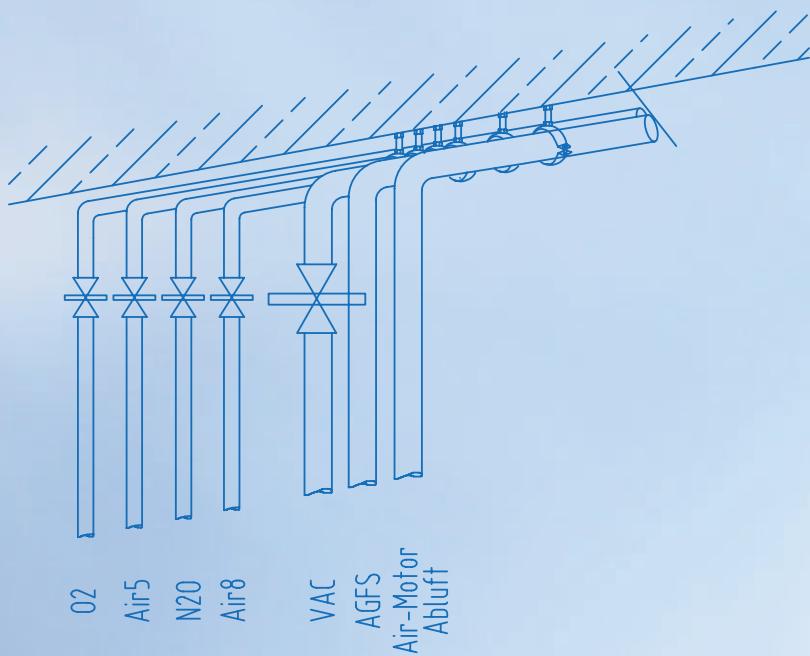
- Case: nickel-plated brass · Sealing: teflon
- Ball: hard-chromed brass with full passage
- Specially degreased and suitable for installation in oxygen supply systems

Artikel-Nr. Article-no.	Größe Size	Anschluss Connection	Länge Length	Gewicht Weight
029-3863	DN 10	G 3/8"	60 mm	180 g
029-3864	DN 15	G 1/2"	65 mm	275 g
029-3865	DN 20	G 3/4"	80 mm	395 g
029-3866	DN 25	G 1"	90 mm	725 g
029-3867	DN 32	G 1 1/4"	110 mm	1.175 g
029-3868	DN 40	G 1 1/2"	120 mm	1.830 g
029-3869	DN 50	G 2	140 mm	3.000 g

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- O-ring sealing screwings made of brass
- Available in different sizes and designs

Design variants:

- Thread on both sides
- Soldered connection on both sides
- Thread on one side / soldered connection on the other side

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- Soldered connection Ø 22 mm
- With O-ring sealing screwing
- Pore size appr. 100 µm
- Pressure range up to 25 bar

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Heyer Aerotech GmbH ■ Nieverner Straße 30 ■ 56132 Nievern ■ Germany